


Using the Basic Ozone Humidifier - 500ml

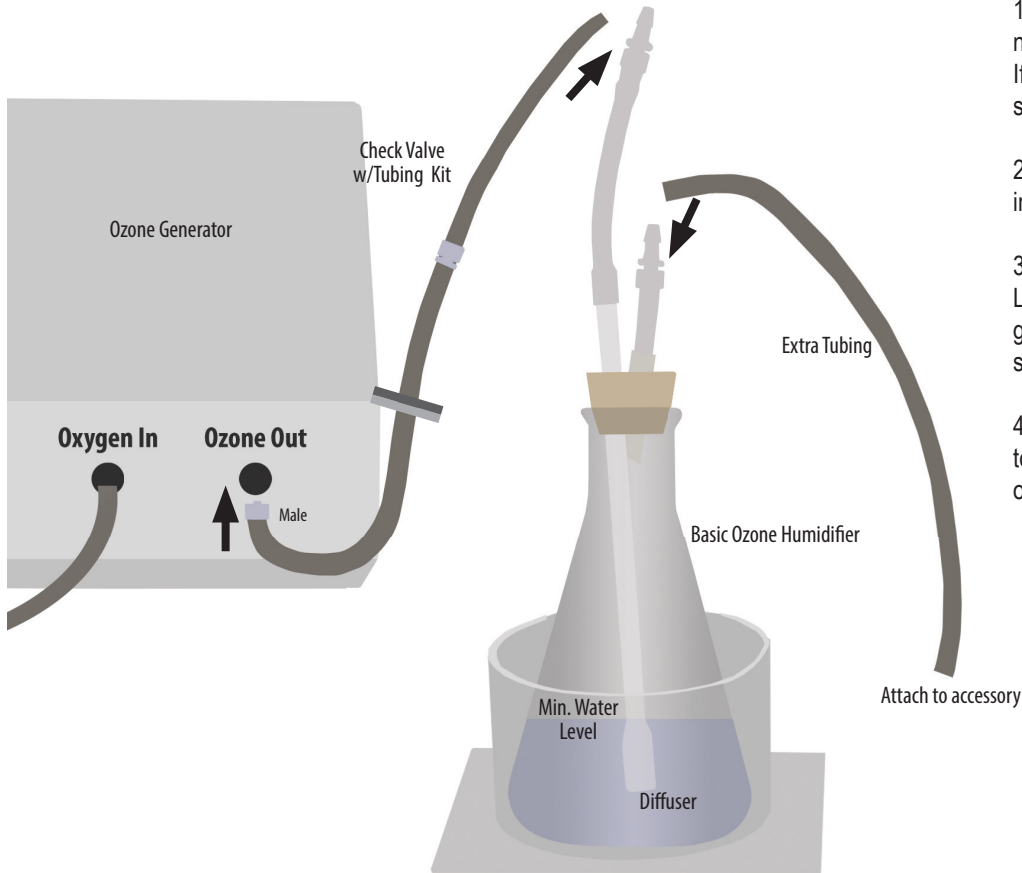


1. Fill the bubbler with at least as much water as is needed to cover the white diffuser (see diagram). If your humidifier came with a stand, place it in the stand.

2. Place the silicone stopper with straws attached into the bubbler.

3. Connect one end of silicone tubing with male Luer lock to the Ozone Out port on your ozone generator. Attach the other end to the water bubbler straw that goes all the way into the bubbler.

4. Attach one end of another piece of silicone tubing to the other straw of the water bubbler. Attach the other end to the accessory of your choice.

If using with an ozone destruct

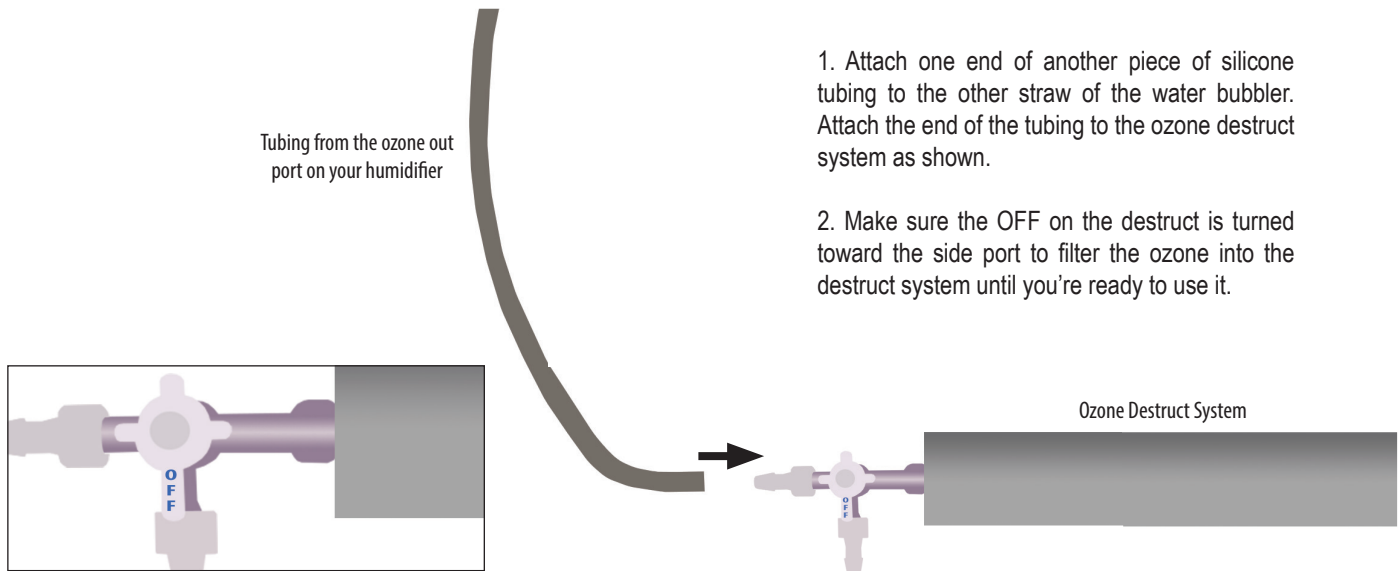

1. Attach one end of another piece of silicone tubing to the other straw of the water bubbler. Attach the end of the tubing to the ozone destruct system as shown.

2. Make sure the OFF on the destruct is turned toward the side port to filter the ozone into the destruct system until you're ready to use it.