


Vaginal Insufflation Device

Direct Connection


1. Connect the Vaginal Insufflation Device to the ozone generator using the check valve kit. If you need longer tubing, you can place it between the check valve kit and the Vaginal Insufflation Device using a tubing connector or a complete Luer lock fitting.

2. When you're ready to begin your insufflation, simply turn on the ozone for the chosen amount of time. We recommend using a humidifier so the ozone will be less drying.

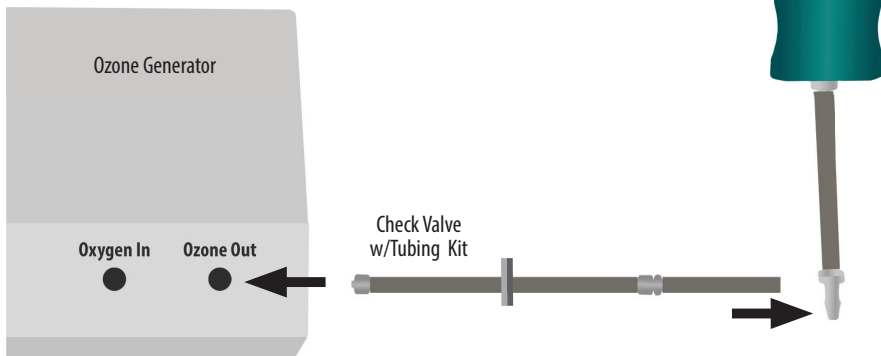

With Insufflation Bag

Fill the bag as shown on the instructions that came with it. Then connect the Vaginal Insufflation Kit to the bag using the Luer lock connectors.

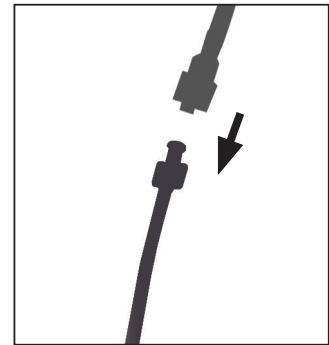


To use the Vaginal Insufflation device with a syringe, please see the instructions that came with your syringe.

With Humidifier and Destruct



1. Use an extra piece of silicone tubing, long enough to reach you, to connect the Vaginal Insufflation Device to the Ozone Destruct System.

2. Make sure the OFF switch on the destruct is pointed toward the insufflation device until you're ready for ozone to flow.

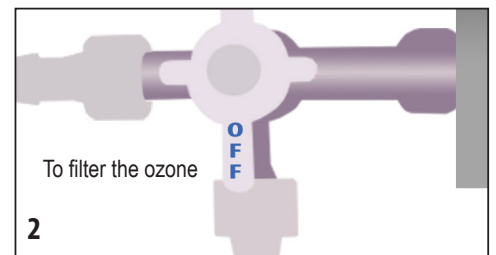

3. When you're ready to begin your insufflation, turn the dial on the destruct toward the destruct system. When you're done, turn the dial back toward the insufflation device (2).

