

Ozone Saline Eye Drop System

1. Place your saline bubbler in one holder and your eyedrop bottle in the other.

[If you have an ozone destruct, attach tubing to it and attach the other end of that tubing to the straw on the bubbler that does **not** go all the way into the jar.]

2. Pour enough saline into the bubbler to cover the diffuser. Use only preservative-free saline. Preservative-free contact solution is ideal for this.

3. Attach tubing from the Ozone Out port on your ozone generator to the straw that goes all the way into the saline bubbler.

4. Turn on your oxygen to 1/8 LPM and your ozone generator to a setting to achieve 50-70 ug/ml. Allow the saline to saturate for about five minutes.

5. Turn off your ozone generator, then your oxygen.

6. Take the stem out of the bubbler and set it aside. Use the eye dropper to pull in saline and apply some drops to your eye.

7. If you want to store excess saline, you can pour it into the eye dropper bottle. You can store it for up to 3 hours.

