

IV DRIP STAND INSTRUCTIONS

Thank you for purchasing the Promolife IV Drip Stand. This stand works for both the 500ml and 1000ml IV Drip Systems. It was designed specifically for Promolife IV Systems and will not work with other units.


This stand can be used on a tabletop or wall-mounted.

TABLETOP

The stand can be used as-is on a table top. For more security, there is a small hole in the bottom you can use to screw the stand in place.

WALL-MOUNTED

To mount the stand, you will first need to unscrew the base and remove it. To do this, remove the four screws above the base.

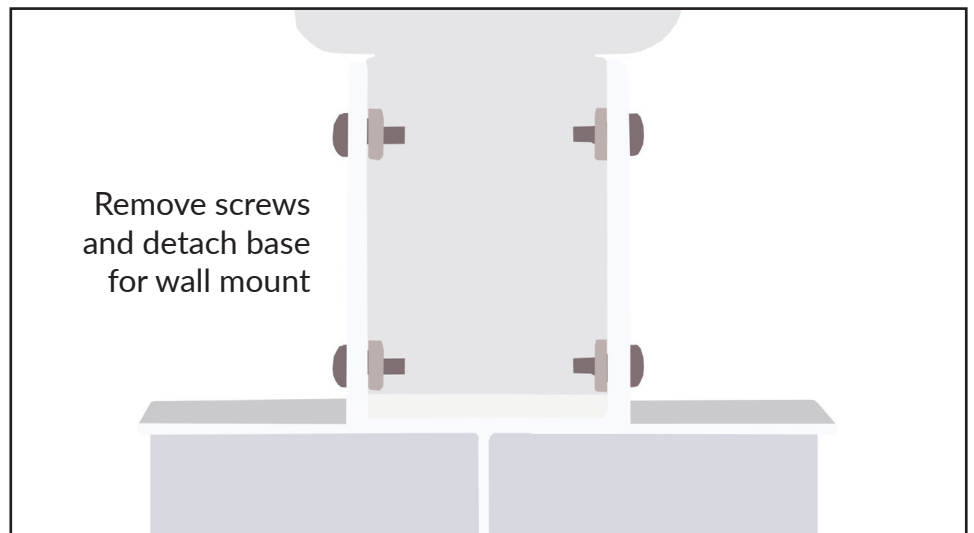

Due to the variation across buildings, mounting hardware is not included. Using the mounting holes in the back of the stand, screw directly into a wall stud or use heavy duty wall anchors. A level is recommended to make sure the stand is properly mounted.

