
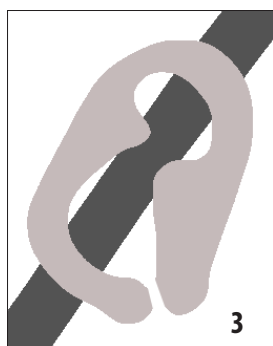


# Using Multi-Chamber Insufflation Bags


## With Humidification

1. Remove the male Luer lock from the destruct system as shown, if it has not been removed already.

2. Screw the fitting on the end of the bag's tubing to the destruct system. Keep the OFF switch pointed toward the bag until you're ready to fill it.

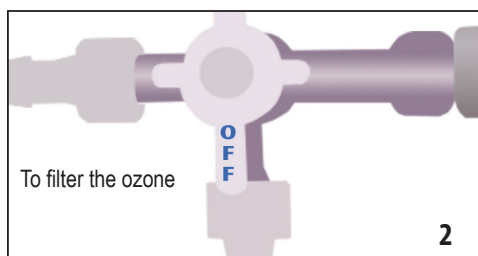


3. Make sure the clip on the bag is open. Turn the OFF switch on the destruct toward the destruct to fill the bag.

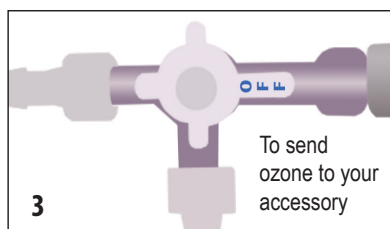


4. When the bag is filled to the desired amount, turn the OFF switch on the ozone destruct back toward the bag and close the clamp on the bag.

5. Unscrew the bag from the destruct.

6. Screw a catheter onto the bag. When you're ready for your insufflation, unclamp the bag and slowly roll it to release the ozone.

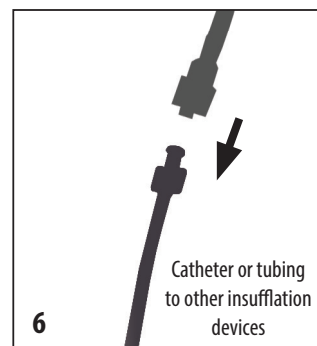

## Without Humidification

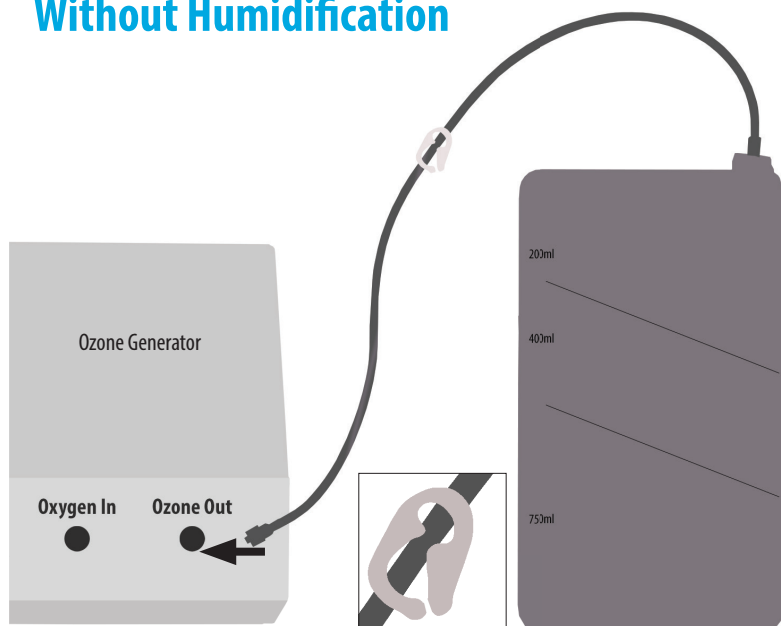

1. Connect the bag directly to the Ozone Out port on your ozone generator. Make sure the clamp on the bag is open.

2. When the bag is filled to the desired amount, remove it from the ozone generator and then clamp the bag.

3. Screw a catheter onto the bag. When you're ready for your insufflation, unclamp the bag and slowly roll it to release the ozone.

